

Reliwash™

More than meets the eye.



* Excludes product and packaging.

TITAN™ rugged, tough and built for the job...

Our new Titan range is designed for the demanding needs of the modern workplace, providing quick and easy access to Reliwash isotonic eyewash solution in the event of eye injury. Titan boxes are 100% recycled and 100% recyclable and are **the latest evolution of our Essentials range.**

EYE-CATCHING

The rugged new heavy duty box design features screen-printed lids. The new graphics stand out from a distance, enabling quick identification in an emergency, especially in a busy workplace. Compact and convenient, the integral bracket allows easy placement and fixing.

EYE-OPENING

Eye injuries can occur in any work environment. Our Eye Wash stations are available in double and triple sizes for all workplace requirements.

Most eye or skin incidents are caused by dust, dirt, metal and wood splinters, and chemicals such as oil and solvents. In these circumstances, rinsing the eye or skin with a solution of 0.9 % sterile sodium chloride will be sufficient. The isotonic solution is suitable for rinsing and cleansing the eyes or skin as a means of first aid in case of injury.



Integral bracket



Rugged corners



Clear identity



Laser etching, no labels



Includes eye bath



and eye pad dressings

Better for the workplace, better for the planet.




+44 (0) 08456 448808 | sales@reliancemedical.co.uk | www.reliancemedical.co.uk

RELIANCE MEDICAL
'Leading First Aid'



REV150223

Easy to use.

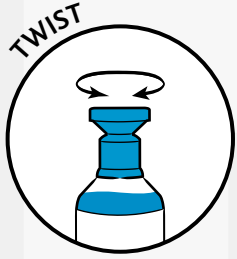

Place the casualty in a comfortable position with their head inclined, so the affected eye is lower than the other eye.



Pour the fluid from the inner corner of the eye outward for 10 seconds at a time, allowing the casualty to blink in between until the object is removed.



If irrigating the eye does not work, cover the injury with a sterile eye pad dressing and seek medical attention immediately. Covering the eye will limit eye movement.

Contaminant:

DIRT, DUST, METAL, WOOD, SPLINTERS

- **Action:** To rinse the eyes and prevent foreign substances coming into contact with them and causing damage.
- **Directions:** Rinse until the substance is removed, or medical help arrives.

Contaminant:

OIL AND SOLVENTS

- **Action:** To rinse the eyes until the unpleasant sensation is over.
- **Directions:** Continue rinsing until you receive medical attention.



EYE WASH STATIONS



955 EYE WASH STATION WITH BRACKET
in Blue Titan Small Box

SIZE: 23cmH x 24.5cmW x 9cmH
UNIT: 1
MOQ: 5

CONTENT:
2 Dressings, Eye Pad
2 Eye Wash, Bottles 500ml



Compatible with AuraPoint brackets



When you see this symbol with our products, you are guaranteed we have reduced plastic in the mould, used 100% recycled materials, 100% recyclable product.



+44 (0) 08456 448808 | sales@reliancemedical.co.uk | www.reliancemedical.co.uk

RELIANCE MEDICAL
'Leading First Aid'

